

Golden Hour®

Platelet Shipper

Holds contents between 20° and 24°C for 24+ hours

GH02AAFC22C

©2006 Minnesota Thermal Science, LLC.
Made in U.S.A. Patents Pending
For complete product information
visit GoldenHourTechnology.com
or call 877-537-9800

1. Remove and precondition TIC™ and lid at 22°C before each use:

Summer Profile: For maximum performance in expected warm ambient conditions cool the TIC slightly to 21°C before loading.

Winter Profile: For maximum performance in expected cold ambient conditions warm the TIC slightly to 23°C.

2. Load TIC with platelets.

3. Close TIC by adding lid and securing straps snugly

4. Place TIC inside box, add top vacuum insulation panel and secure latches.

