

! WARNING !

FOR YOUR SAFETY, READ ALL INSTRUCTIONS AND WARNINGS CAREFULLY BEFORE INSTALLING AND USING YOUR EASY ACCESS TOILET SEAT!


- DO NOT STAND ON SWING ARM IN THE EXTENDED OR CLOSED POSITION AS THE ARM CAN MOVE AND CAUSE POSSIBLE INJURY.
- DO NOT PLACE EXCESSIVE WEIGHT ON THE SWING ARM WHILE IN THE EXTENDED POSITION AS THIS MAY CAUSE POSSIBLE DAMAGE TO THE SEAT AND POSSIBLE INJURY.

Thank you for purchasing the EASY ACCESS Toilet seat. Please check your accessory kit for the following items before installation:

ITM	QTY	DESCRIPTION
A	1	Easy Access Toilet Seat
B	2	Seat Hinge Mounting Bracket
C	2	Rubber Grommet
D	2	Nylon Toilet Seat Nut

INSTALLATION INSTRUCTIONS:

1. Remove and discard old toilet seat.
2. Clean all surfaces between lower and upper hinge bracket mounting holes.
3. Insert smooth portion of hinge mounting bracket on each side of seat hinge holes.
4. Guide rubber grommets onto each threaded portion of seat hinge bracket.
5. Guide each threaded portion of hinge bracket onto the toilet seat mounting holes.
6. Attach each nylon toilet seat nut to threaded portion of hinge mounting bracket.
7. Tighten each nut thoroughly making sure not to over tighten or strip nut thread.
8. Check seat for excess movement and tighten nuts if necessary.

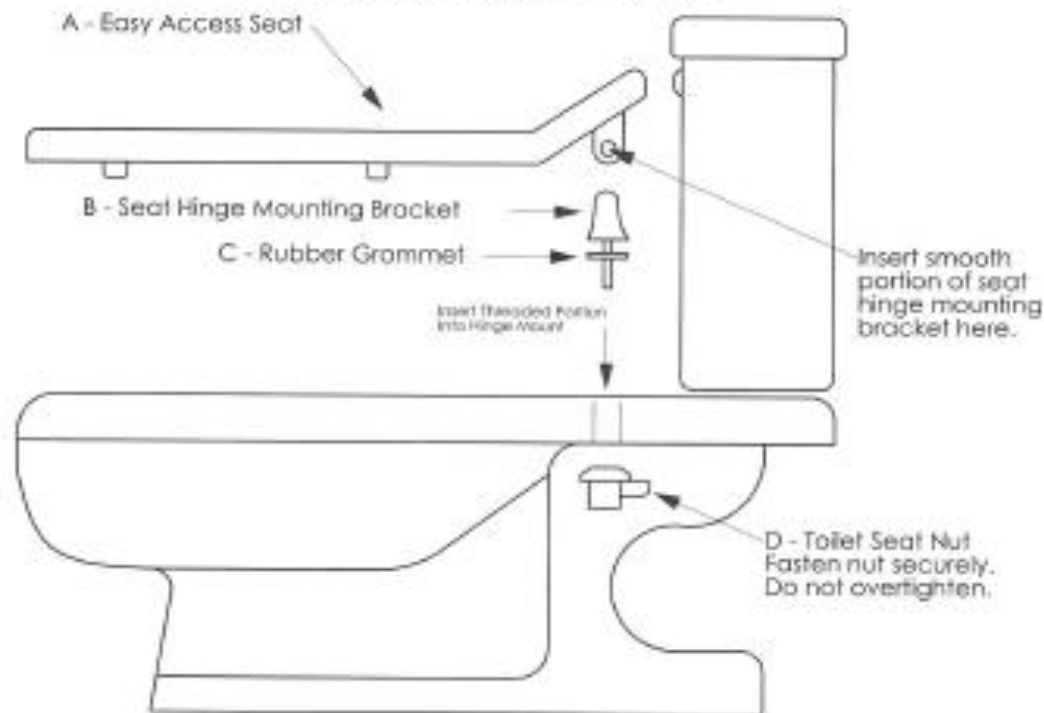

The Adjust for Comfort Toilet Seat™