

# UltraClenz

Patient Safeguard System™



Caregiver sanitizes/washes hands making the badge turn **GREEN**. The Dispenser Beacon and Badge communicate with each other. The information is then sent from the Dispenser Beacon to the Bentley™ Wireless Communication System.



The floor supervisor is notified realtime of the caregiver's hand hygiene status on a computer or tablet device.

Non-compliant events are highlighted in red.



The Badge turns **YELLOW** when the caregiver approaches the patient's bed (enters proximity boundary) as a cautionary reminder that the caregiver has been in contact with a patient.



The Bed Beacon sends a message to the Bentley™ Wireless Communication System advising that the caregiver has been in contact with the patient. The message also includes the caregiver's Badge status.

If the caregiver leaves the patient area for several minutes and returns back to the same patient OR if the caregiver approaches a different patient without sanitizing/washing their hands, the Badge turns **RED**.



A repetitive audible alert will activate on the Badge while the Badge is in a non-compliant or **RED** state.



The non-compliant event data is then sent to the Bentley™ Wireless Communication System.

If you want Clean..... Let us give you a Hand!

1.800.931.8911 [www.ultraclenz.com](http://www.ultraclenz.com) [info@ultraclenz.com](mailto:info@ultraclenz.com)