



CellerateRX is an amazing fast-acting gel that delivers the power of activated collagen to your body immediately, thus jump-starting the wound-healing process. It's affordable and perfect for most wounds, including surgical wounds, 1st- and 2nd-degree burns, cuts, gashes, blisters, diabetic ulcers, skin tears, and more. Even the most troublesome wounds are easily managed.

CellerateRX can be used in all stages of the wound-healing process. Simply clean the wound, apply CellerateRX gel, then cover with a bandage.

CellerateRX provides results often twice as fast as Mother Nature alone, lessens the potential for scarring, and even helps reduce any pain in the process.

FDA Approved

Cellerate[®]_{RX}
ACTIVATED COLLAGEN™ (CRX_α)

www.WellDyneHealth.com



First-time application of CellerateRX[®]:

1. The wound should be thoroughly cleaned before applying CellerateRX[®].
2. Apply CellerateRX[®] to the wound. CellerateRX[®] gel should cover the entire wound, including the edges.
3. Cover the CellerateRX[®] and wound with a bandage or dressing that keeps the wound moist.

Reapplying CellerateRX[®] to the wound:

1. Gently remove the bandage or dressing from the wound. If the bandage or dressing sticks to the wound, moisten it before removing.
2. Do not clean or remove CellerateRX[®] from the wound.
3. Reapply CellerateRX[®] 2-3 times per week or as needed. Repeat steps 1-3 above.

Tunneled or undermined wounds:

1. Follow Steps 1-3 in the Reapplying instructions. CellerateRX[®] may be drawn up into a syringe and applied into the tunnel or undermined area through the syringe.
2. The CellerateRX[®] does not need to be removed from the wound on reapplication, as it is biocompatible.

Skin flaps or grafts:

1. Apply CellerateRX[®] to the underside of the skin flap and the wound, and then flip the skin flap back onto the wound.
1. A transparent film can be placed over the flap and left in place for 2-4 days.

Burns:

1. Follow Steps 1-3 in the Reapplying instructions.
 2. Collagen occludes nerve endings to reduce pain.
- 