



THE MIGHTY LITTLE BITE-PLANE
QuickSplint[®]
With Jaw-Dropping Powers



PRODUCT INFORMATION

Provided by QuickSplint LLC

Interim overnight dental splint for use up to 4-6 weeks

- First Aid for Acute Symptoms
- Protective & Stabilizing Splint
- Diagnostic Aid
- Deprogrammer
- Transitional Appliance
- FDA 510k for use by a licensed professional



Dozens of practical applications across dentistry!

- General Dentistry
- Cosmetic / Restorative Dentistry
- Periodontics
- Prosthodontics
- Endodontics
- Oral Surgery
- Orofacial Pain

Ease of fabrication is a major advantage of QuickSplint

- Ready in 5 minutes
- Comfortable for patient
- Use heavy body VPS (your brand or QuickSplint brand)
- DA can fabricate chairside

Fill



Use fast-set heavy body VPS.

Align



Align on upper or lower arch while patient bites down.

Trim



Trim with scalpel when set. QuickSplint is ready to wear!

See instructional videos online at: www.quicksplint.com

QuickSplint works like an Anterior Bite Plane



<http://www.quicksplint.com/quicksplint-videos/#>

Click link to view animation courtesy of QuickSplint & Anomalous Medical



FOR GENERAL DENTISTRY

The Interim Dental Splint with Dozens of Applications across Dentistry

✓ **The First Easy to Use Temporary Dental Splint**
Provide immediate pain relief for jaw pain or headache, protect new restorations from nighttime occlusal forces. QuickSplint is an all-around protective and stabilizing dental splint designed to be worn for up to 4 to 6 weeks.

✓ **Diagnose and Treat TMD**
Use QuickSplint for differential diagnosis of sprain/strain or other conditions of the jaw, joints and muscles.

✓ **Minimize Complications**
Provide QuickSplint the day of dental procedures to inhibit overnight parafunctional clenching and prevent possible complications associated with micro-trauma, trismus and jaw strain.



QuickSplint is an all-around protective and stabilizing dental splint and an indispensable diagnostic aid.



Use QuickSplint in restorative case planning, to protect new dental work, minimize complications, and promote healing.

FOR RESTORATIVE / COSMETIC DENTISTRY

Case Planning Aid. Interim Protection & Healing Aid. Transitional Appliance

✓ Restorative Case Planning Aid

Diagnose parafunction and engage the patient in necessary conversation about the possible impact on the restorative case.

✓ Simplify Deprogramming

Swift and handy aid for taking centric bite relations or final equilibration.

✓ Protect Temporaries & New Restorations

Provide QuickSplint as a healing aid and an extra measure of care to protect dentition and allow the jaw to relax overnight.

✓ Transitional Appliance

Use QuickSplint as a temporary protective appliance until a custom appliance is delivered.



QuickSplint is an easier way to provide a periodontal splint and it is ideal and practical for most cases.

FOR PERIODONTICS

Minimizes Complications Prior to, During, and Following Periodontal Surgery

✓ **Easy Periodontal Splint & Post Operative Healing Aid**

For periodontal surgery, and other soft- and hard-tissue surgeries use QuickSplint to stabilize teeth and protect gingival tissues post-operatively. Provide with our specialized splint instructions for perio and implant post-operative use.

✓ **Implant Placement**

Provide QuickSplint the day of surgery to inhibit parafunctional clenching and minimize complications during the implant healing phase.

✓ **Address Jaw Fatigue**

During long procedures, QuickSplint is prescribed to minimize complications prior to, during, and following long procedures.



Provide QuickSplint the day of surgery to inhibit parafunctional clenching and minimize complications during the implant healing phase.

FOR DENTAL IMPLANTS

Solves the Problem of Practical Protection for Newly Placed Implants and Promotes Healing

- ✓ **Aid in Case Planning & Implant Selection**
Gauge the presence and severity of parafunction prior to selection of implant system.
- ✓ **Post Operative Healing Aid**
Provide QuickSplint the day of surgery to inhibit parafunctional clenching, minimize complications, and promote healing.
- ✓ **Address Jaw Fatigue**
Use QuickSplint to minimize jaw and muscle sprain/strain prior to, during and following long procedures.
- ✓ **Transitional Appliance after New Implants**
Use QuickSplint as a temporary protective appliance until a custom orthotic is delivered.

THE MIGHTY LITTLE BITE-PLANE
QuickSplint[®]
With Jaw-Dropping Powers



Use QuickSplint for determination of myogenous versus odontogenic pain. QuickSplint can also help preserve tooth structure and save crowns!



FOR ENDODONTICS

Accurately Diagnose Endodontic Patients and Prevent Pain

✓ **Endodontic Diagnosis**

Use QuickSplint for determination of myogenous versus odontogenic pain.

✓ **Preserve Tooth Structure**

Use QuickSplint immediately after therapy to avoid having to cut down the occlusion on the tooth or crown.

✓ **Post Treatment Healing Aid**

Provides patient comfort overnight by reducing bite-force and giving the instrumented tooth time to heal from apical inflammation. Allows pericranial muscles to relax.

✓ **Address Jaw Fatigue**

Use QuickSplint to minimize jaw muscle strain/sprain prior to, during, and following long procedures.



Wear marks from clenching and bruxing are recorded on the QuickSplint.

FOR JAW PAIN & TMD

The Conservative First Step in the Diagnosis & Treatment of Jaw Pain & TMD

✓ **Treat Acute and Urgent Pain**

Use QuickSplint to initiate conservative treatment of TMJ-related pain. QuickSplint provides overnight relief from acute onset muscle and/or joint symptoms of bruxism, and TMD or muscle pain.

✓ **Diagnose Parafunction & Bruxism**

Use QuickSplint as a diagnostic aid for parafunction & bruxism. QuickSplint records wear marks.

✓ **Transitional Appliance**

Use QuickSplint as a temporary protective appliance until a custom orthotic is delivered.



Provide QuickSplint the day of surgery to minimize complications and promote healing.

FOR ORAL SURGERY

Helps Oral Surgeons Appropriately Respond to Jaw Joint and Muscle Sprain/Strain Injuries

✓ **Treat Jaw Sprain/Strain**

Use QuickSplint and the JAMSS protocol to treat jaw joint and muscle sprain/strain injuries that are common in oral surgery.

✓ **TMD Prevention**

Use QuickSplint and the JAMSS protocol to help minimize jaw muscle strain/sprain and prevent TMD related to oral surgery.

✓ **Post Operative Healing Aid**

Provide QuickSplint the day of surgery to inhibit parafunctional clenching, minimize complications, promote healing, and to comfort the patient.



Patients usually experience rapid relief of their symptoms using QuickSplint in combination with Migracises jaw exercises.

FOR HEADACHES

QuickSplint and Migracises -- A Powerful Combination to Help Patients with Headaches


✓ **Treat Headache Pain**

Patients suffering from tension headaches, episodic headaches, and chronic tension headaches usually experience rapid relief of their symptoms using QuickSplint in combination with Migracises.

✓ **Help Patients**

Avoid Daytime Clenching

Migracises are a set of jaw exercises designed to help patients avoid daytime clenching or setting of teeth which causes jaw and temple muscles to overwork and become sore.

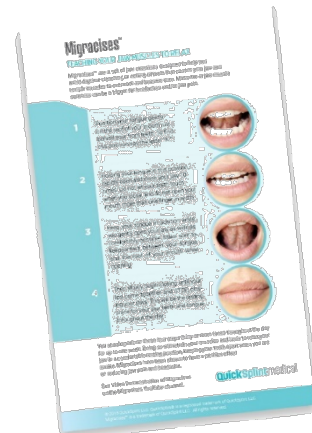



FDA 510k-cleared for these indications:

- Temporary protection of teeth and restorations from injury due to bruxism or clenching.
- Temporary relief of Temporomandibular Joint Disorder (TMD) and bruxism by reducing muscle tension.
- Temporary treatment of TMD along with the relief of associated headaches and pain.



QuickSplint is packaged with patient instructions in its own container for easy transport and storage.

QUICKSPLINT WAS DEVELOPED BY AN OROFACIAL PAIN EXPERT



Bradley Eli, DMD, MS

Bradley Eli DMD, MS is nationally recognized as an expert in orofacial pain and sleep disordered breathing by both physician and dentist colleagues, and his practice is devoted solely to these disorders. He is one of less than 100 doctors in the US with this unique training which includes a post-doctoral master's degree at UCLA and over 25 years of clinical experience in pain management and sleep disorders.

QuickSplint was developed for his pain practice to provide patients with a conservative, immediate device, while his differential diagnosis is developed. Over 1200 patients have benefitted from QuickSplint use over a 10 year period.



www.quicksplint.com
800-760-0526



QuickSplint® Combo Pack

Splint & VPS bundle offers customers an extra \$16 savings as compared to purchasing products separately.

- 12 splint units with take-home containers
- 2 cartridges QuickSplint VPS – 50ml with 12 mixing tips

\$260

QTY

1

Reimbursable – D8210 Removable appliance therapy;
D9940 Occlusal Guard to minimize effects of bruxism and other occlusal factors.

THE MIGHTY LITTLE BITE-PLANE

QuickSplint®



With Jaw-Dropping Powers

FDA APPROVED FOR USE WITH TMD AND BRUXISM PATIENTS OR FOR PROTECTION OF DENTAL RESTORATIONS.

© 2016 QuickSplint LLC. QuickSplint® is a registered trademark of QuickSplint LLC. All rights reserved.

www.quicksplint.com

800-760-0526